



Fig 1

**INFOTRONIC ULTRAFLEX GONIO
MANUAL UPDATE:**

**New Gonio fixation, it's a real
sensation.....**

**This is the newest version of Gonio sensor
fixation for the INFOTRONIC ULTRAFLEX
GAITANALYSIS SYSTEM.**

With this strap fixation based on velcro band the
fixation of goniometers is completed in just **a few
seconds!**

**Left you see (fig 1) the goniometer fixated
in the two straps.**

**You can see that the sensor and the
amplifier are fixed together and so the
small connectors cannot come loose.**

**This fixation of the goniometer in the
straps you have to do only once.
After that, you just put on or remove the
whole assembly of fig. 1 by the velcro
straps**



**HIP-KNEE-ANKLE
FIXATION:**

To measure the hip angle we made a special
shortpant which is extremely flexible.

You can even wear it **over your pants** if you
prefer (not recommended but possible) It is
better to wear it only over your underwear!

On the shortpant some velcro strips have
been sewn, so you just put the gonio sensors
exactly on the strip and all is done !!!! (red
side onto the pantside.)

**The fixation on the knee and ankle are
done in the same way (see pictures).
This is very easy and saves a lot of time
compared with the old adhesive tape
method!**

